

Aptima® Performance Data on the Panther® System

Specimen Type		Aptima Combo 2®				Aptima® Trichomonas vaginalis		Aptima® HSV 1 & 2			
		Chlamydia ¹		Gonorrhea ¹		Trichomoniasis ²		Herpes simplex virus ³			
		Sensitivity	Specificity	Sensitivity	Specificity	Sensitivity	Specificity	Sensitivity		Specificity	
	Multitest Swab Clinician Collected Vaginal Sample	97.2%	98.5%	97.7%	99.6%	100.0%	98.2%	*		*	
	Multitest Swab Patient Collected Vaginal Sample	97.2%	98.5%	97.7%	99.6%	*	*	*		*	
	Multitest Swab Anogenital Skin Lesion Sample	*	*	*	*	*	*	HSV-1	HSV-2	HSV-1	HSV-2
		94.7%	98.4%	99.6%	92.8%						
	Female Endocervical Swab	97.2%	99.3%	100.0%	99.8%	100.0%	98.1%	*		*	
	Male Urethral Swab	100.0%	99.1%	100.0%	100.0%	*	*	*		*	
	Urine (Male)	95.2%	99.8%	98.7%	99.7%	*	*	*		*	
	ThinPrep® Pap Test Vial	98.2%	100.0%	100.0%	100.0%	100.0%	98.6%	*		*	

*These sample types are not FDA-cleared for the specific assays listed.

References: **1.** Aptima Combo 2 Assay [package insert]. 502446-IFU-PI, Rev. 002. San Diego, CA: Hologic, Inc.; 2016. **2.** Aptima Trichomonas vaginalis Assay [package insert]. 503684-IFU-PI, San Diego, CA: Hologic, Inc.; 2011.

3. Aptima HSV 1 & 2 Assay [package insert]. AW-15636-001. San Diego, CA: Hologic, Inc., 2017.

TRN-00117-001 Rev. 004 © 2017 Hologic, The Science of Sure, Aptima Combo 2, Aptima, Panther, ThinPrep, Tigris and DTS are trademarks and/or registered trademarks of Hologic, Inc., and/or its subsidiaries in the United States and/or other countries.